

PPE COVID-19

PRODUCT LEAFLET

PRODUCT CODE 8913-TY2-17
DESCRIPTION Medical face mask disposable Type I
UNIT OF MEASURE 50 pieces



DIMENSIONS 17.5 x 9.5 cm

PRODUCT DESIGN AND SPECIFICATION The product consists of mask body, mask band, and nose clip. Mask is non-sterile and disposable.

Characteristic	Result
Size	
<ul style="list-style-type: none">LengthWidth	<ul style="list-style-type: none">17.5 cm9.5 cm
Fabric weight	
<ul style="list-style-type: none">Outer layer non-wovenMiddle layer melt blownInner layer non-wovenEar loop polyester	<ul style="list-style-type: none">Min 25 g/m²Min 25 g/m²Min 25 g/m²Min 35 g/m²
Nose wire length	10.5-10.6 cm
Strength of connecting point of tie or elastic to body	Min 11N
Flame retardation	3-4 sec
Bacterial Filtration Efficiency (BFE)	95-96%
Differential pressure	NMT 33 Pa/cm ²
Microbial cleanliness	4~10CFU/g

INTENDED USE Medical face mask Type 1 should only be used for patients and other persons to reduce the risk of the spread of infections particularly in epidemic and pandemic situations. The Type 1 mask is not intended for use by healthcare professionals in an operating room or in other medical settings with similar requirements.

INSTRUCTION FOR USE

- Hold the mask by both ear loops and place a loop around each ear
- Mould the bendable metallic upper strip to the shape of the nose by pinching and pressing it down with your fingers
- Pull the bottom of the mask over the mouth and chin
- Be sure the mask fits snugly
- Do not touch the mask once in position

DISPOSAL Dispose as medical waste in accordance with hospital, administrative and/or local government regulations and policies.

REGULATORY APPROVAL EU medical Device directive (Declaration of Conformity as Per MDD 93/42/EEC Risk class I)

APPLICABLE
STANDARDS

Product is complying with the following standards:

- EN14683:2019 (medical face masks - Requirements and test methods)
- ISO 10993 (biocompatibility)
- EN ISO 11737 (test method microorganisms)

STORAGE

Store in a well ventilated and clean indoor storage, without corrosive gases. It is strictly forbidden to store the products in a warehouse where chemical and wet items are stored as well.

SHELF LIFE

36 months

PACK SIZE

50 pieces per box

PRODUCT PHOTO

