

PPE COVID-19

PRODUCT LEAFLET

PRODUCT CODE	H625-TY2-17
DESCRIPTION	Collection and Transport Kit VTM, for oral or nasal sampling
UNIT OF MEASURE	50 pieces



DIMENSIONS	Not applicable
PRODUCT DESIGN	Citoswab Collection and Transport Kit provides safe and convenient collection and transport of clinical specimens. Each Collection and Transport Kit comprise a sterile peel pouch containing a sterile flocked swab used to collect the sample, a 10 ml transport tube containing 3 ml VTM medium into which the swab is placed after sampling, and a biohazard specimen bag used to place the transport tube in for transportation.
INTENDED USE	Citoswab Collection and Transport kit is a ready to use system used for collection and transport of clinical specimens.
INSTRUCTION FOR USE	<ol style="list-style-type: none">1. Peel off the pouch and collect sample2. Put the swab into the medium3. Snap the stick of swab4. Screw the tube cap tightly5. Label the specimen6. Put the tube into biohazard bag
DISPOSAL	Dispose as medical waste in accordance with hospital, administrative and/or local government regulations and policies.
REGULATORY APPROVAL	Collection swab: Approved in EU (CE) as per EU Medical Device Directive (CE Certificate as per MDD 93/42/EEC, risk class IIa) Collection Tube and Biohazard Bag: Approved in EU (CE) as per EU In Vitro Diagnostic Directive (declaration of conformity as per IVD 98/79/EC, risk class IVD Others). Manufacturer quality management system as per ISO 13485:2016 for medical device industry.
APPLICABLE STANDARDS	Product is complying with the following standards: Product is as per CLSI-M40A (for quality control of Microbiology specimen Transport Device) Product is also suitable for Performing Molecular Biology Technique
STORAGE	Store between 5°C to 25°C and keep in dry place
SHELF LIFE	12 months
PACK SIZE	50 tests per box 200 pieces per shipper carton

